

PHILIPS

DreamStation 2

CPAP Advanced

Auto CPAP Advanced

Provider training guide



Flexible setup

Choose an out-of-the-box, no-touch setup¹ or a guided, streamlined, low-touch setup on the device.

1 No-touch setup

For no-touch setup, accept data-driven defaults or personalize remotely with Care Orchestrator Patient Management System.

Personalizing prescriptions remotely:

1. Log into Care Orchestrator to initiate the setup process.
2. Add the required patient information, the device serial number, and the patient's prescription.
3. Select “**Save**” to automatically push the prescription to the device. When the device is plugged in and powered up, it will initiate a modem call to receive the new prescription.

Device defaults:

Mode: AUTO

Pressure: MIN = 4 cm H₂O and MAX = 20 cm H₂O

FLEX: 3

Humidification: 3

Heated tube temperature: 3

Tube size: 15mm standard (device will automatically detect the heated tube size when it is attached)

Mask resistance: X1

Ramp Plus time: 30 minutes


Ramp Plus pressure: OFF

Opti-Start: enabled

AUTO ON: enabled

Patient controls access: OFF

Patient data access: OFF

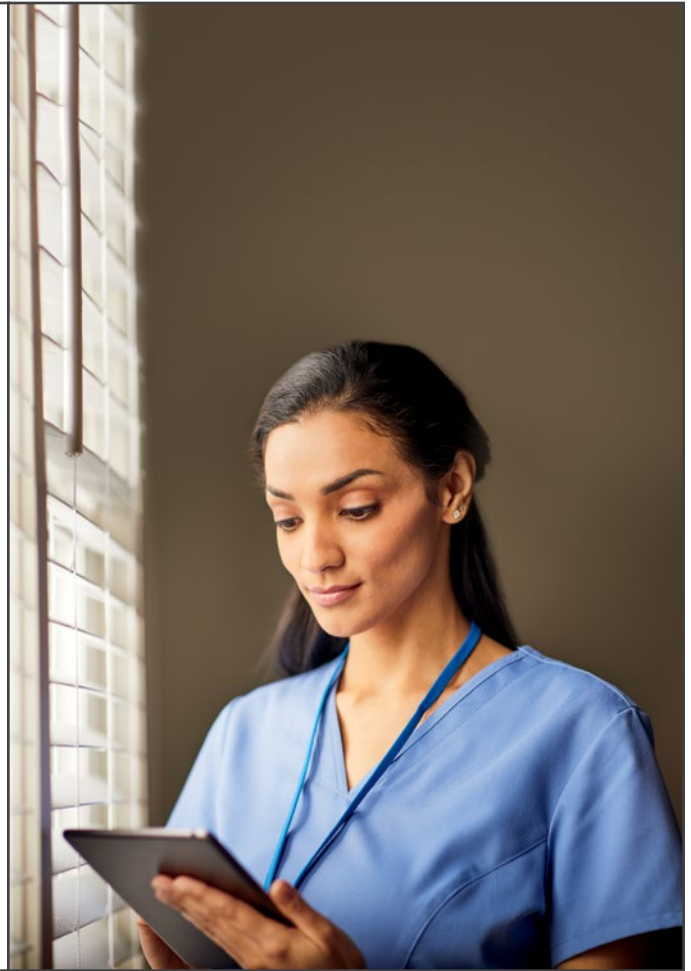


2 Low-touch setup on the device

Entering Provider mode

Prescriptions are set via Provider screens that are unavailable to patients.

With the device plugged in, press and hold the **Therapy button** for several seconds. When the light ring around the **Therapy button** begins flashing, continue to hold it down while using another finger to swipe down on the screen until “**Entering Provider Mode**” appears.

Note: In some instances, you will need to set the language and time when the device is plugged in for the first time.

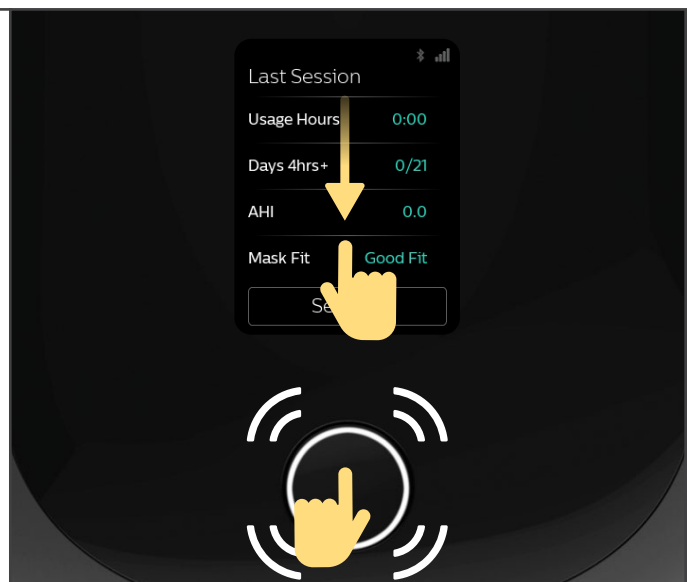

¹Assumes acceptance of smart default settings. Alternatively, you can set up prescriptions entirely via Care Orchestrator with no device touch.


Setting a prescription on the device

Upon first entry into Provider mode, you will be prompted to select either “Start Setup” or “Use Defaults.”

If you do not want to use the defaults, you will be guided through a series of prescription screens that will allow you to make changes to the mode, pressure, FLEX, and tube size.



1 Mode: CPAP or AUTO

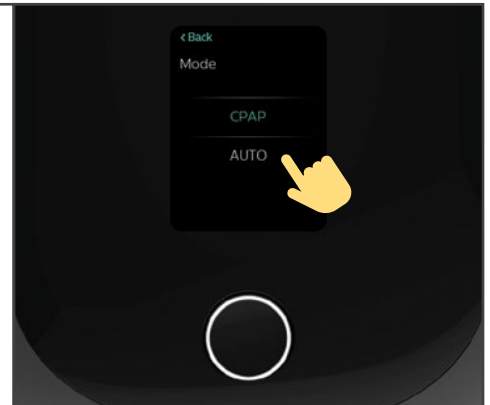


2 Pressure(s): 4 to 20 cm H₂O

Use the + or - symbols to select the prescription pressure. Tap “Confirm” to save each setting.


For AUTO mode, “Max Pressure” will appear after “Min Pressure” has been confirmed.

Note: Pressing down on + or - will allow you to advance through the numbers quickly.

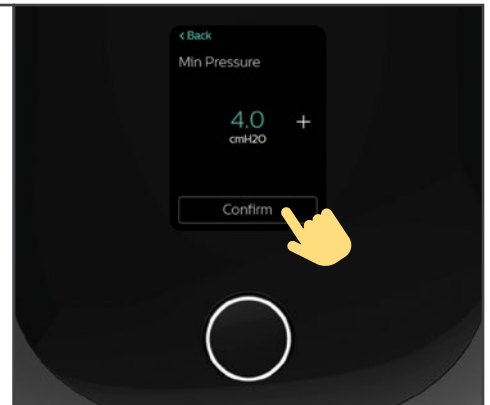

3 FLEX: Off, 1, 2, or 3


The **FLEX setting** offers the patient pressure relief during inspiration (in AUTO mode) and expiration.

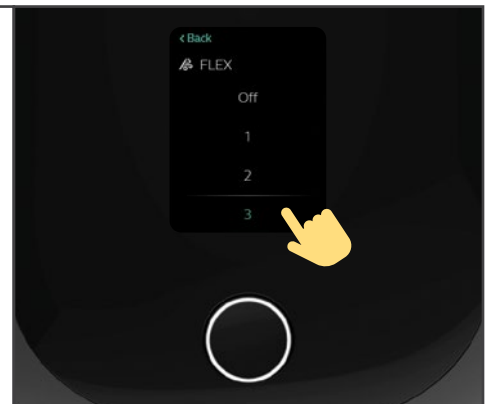

4 Tube size: 12mm, 15mm, or 22mm

If a heated tube is connected during setup, the device will automatically detect it and skip the tube size screen.

Once complete, you will be taken to the Provider menu and see a summary of Therapy settings.

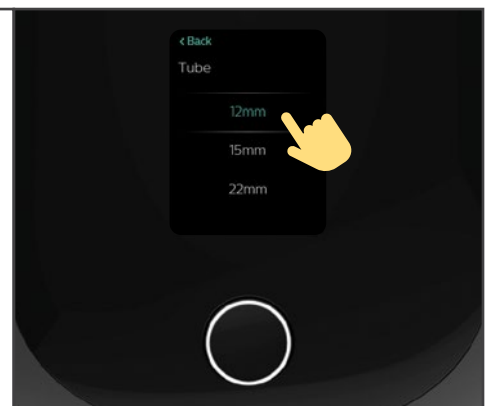

Navigating the Provider menu

Additional settings and information are organized into three basic tabs: Therapy, Data, and Device. Dots indicate how many additional screens are available. Swiping up or down enables navigation between screens. Tapping on information in blue allows you to change a setting.

Therapy tab

Settings

Mode: CPAP or AUTO

Pressure: 4 to 20 cm H₂O


FLEX: Off, 1, 2, or 3

Tube size: 12mm, 15mm, or 22mm

Ramp Plus allows a patient to set a comfortable starting pressure while falling asleep.

• **Time:** 15, 30, or 45 minutes

• **Pressure:** Off or 4 to 10 cm H₂O

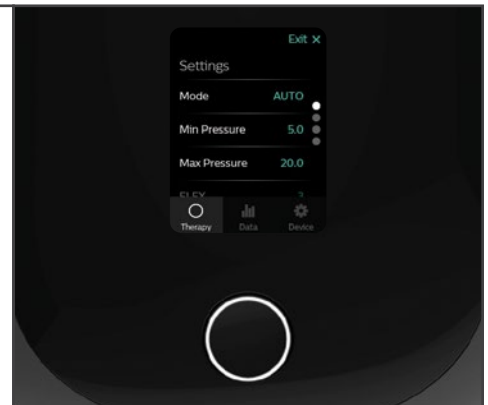


Patient Access

Patient Controls allows you to enable or disable patient visibility of settings when therapy is turned off (Ramp Plus, humidification, heated tube temperature, FLEX, and time).

Note: Patient will always have access to Ramp Plus pressure, humidification, and heated tube temperature (if using heated tube) while therapy is on and running.

Patient Data allows you to enable or disable patient visibility of AHI and View Reports.

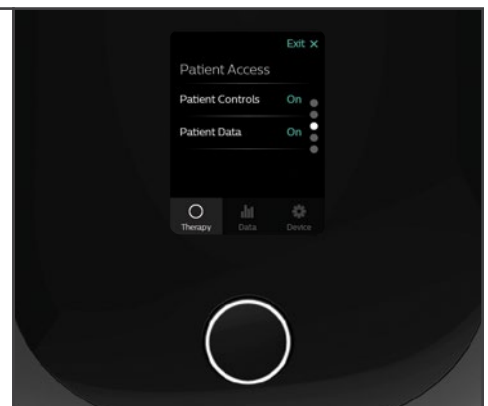

Optional settings

EZ-Start reduces the therapy pressure setting for the first few days of operation and gradually increases the setting until the prescription therapy pressure is reached.

CPAP Check delivers a fixed CPAP pressure while automatically adjusting the pressure level to meet patient needs over the long term.

Mask Type allows selection of the appropriate resistance setting for a Philips mask to adjust the level of pressure compensation. Refer to the mask packaging to identify the resistance setting: X0, X1, X2, X3, X4, X5.



Auto Trial


Auto Trial enables the device to deliver Auto CPAP for a selectable number of days of patient use.*

Duration: Off, 3, 7, 14, 21, or 30 days

Min Pressure: from 4 cm H₂O to Auto maximum pressure

Max Pressure: from Auto minimum pressure to 20 cm H₂O

*Applies to CPAP mode only.

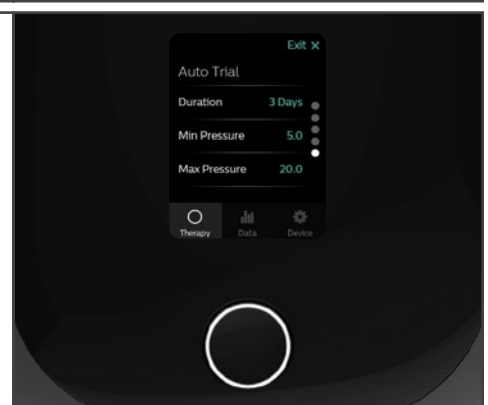

Data tab

30-day data

Average Usage shows hours and minutes of average therapy time.


Average AHI shows average of individual AHI values from each session.

Days 4hrs+ displays the total number of sessions lasting 4+ hours out of the 21 sessions needed for compliance.

Average Mask Fit displays the average mask fit percentage.

90% Pressure displays the average pressure values for individual sessions with 90% pressure.*



*Applies to AUTO mode only.

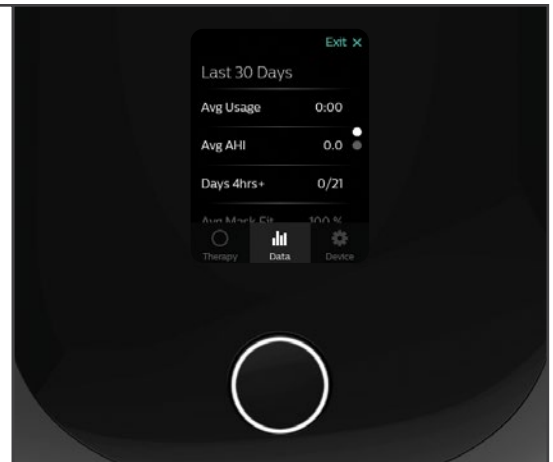

Data tab

Detailed data

Select “**View Reports**” to see additional patient data for customizable timespans.

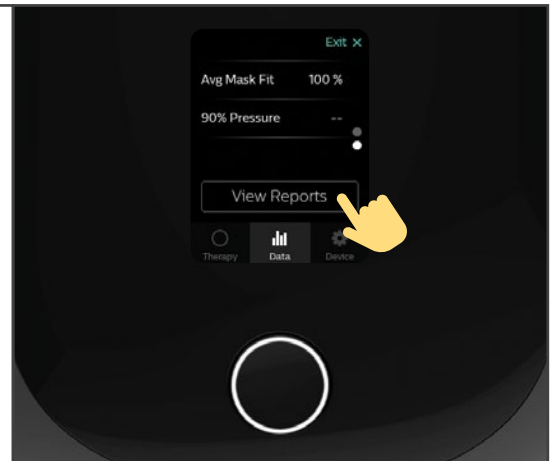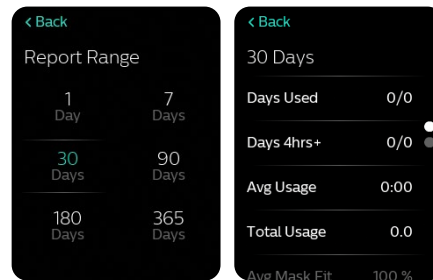

Device tab



Use this tab to perform a device reset, change language and time, see machine hours, view serial number and software version, and set a PIN.



Exiting Provider mode

To leave the Provider menu at any time, tap “**Exit**” and then “**Yes**” to confirm.

Once you return to Patient mode, the device is ready for use.

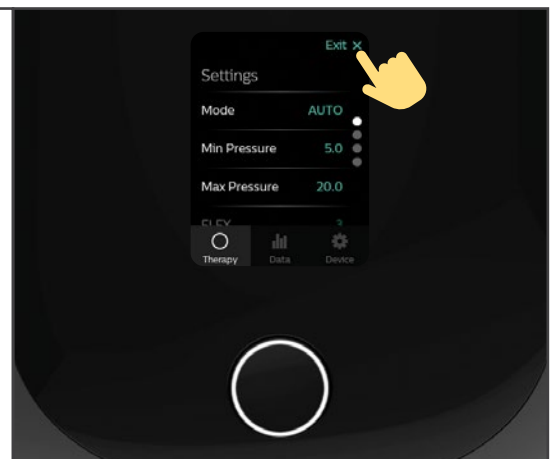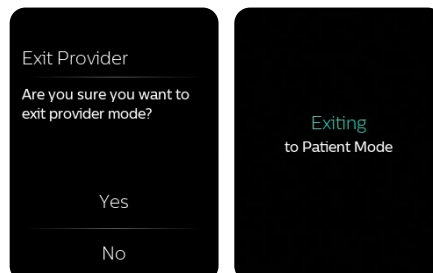



© 2021 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.

www.philips.com

Caution: U.S. federal law restricts these devices
to sale by or on the order of a physician.

RRDPGH EB 1/21 PN 1149713 MCI 4110632 v00
1010 Murry Ridge Lane, Murrysville, PA 15668
800 345 6443 • 724 387 4000