

Accessory Cleaning and Inspection Instructions

Before resuming therapy on your new or replacement machine, you or your equipment provider must **visually inspect the following accessories** for evidence of particulate contamination:

- Tubing
- Mask components (mask, cushions and integrated tubing)
- Humidifier tank and seals (if applicable)

If visible particles are evident on any of these components:

- discard components and contact your equipment provider for replacement.

If there is no evidence of particles on any of these components:

- clean each component per their respective instructions for use
- reassemble and resume therapy on your new device.

