

WRIST BRACE

APPLICATION:

1. Undo both the contact closures straps.
2. Place hand into the support and ensure stockinette is positioned on top of the hand.
3. Align the aluminum palmer stay to the palm of the hand. Adjustments on the aluminum palmer stay can be performed by a certified medical professional.
4. Fasten the adjustable contact closure strap and secure to a snug fit. Adjustments may be necessary as hand, wrist and thumb swelling patterns change.
5. Fasten webspace contact closure strap across aluminum dorsal stay.
6. Adjust for comfort.

WASHING INSTRUCTIONS

- This product may be cleaned using a mild detergent and cold water.
- Aluminum palmer stay should be removed before washing.
- Do not bleach or use strong detergent.
- Do not wash in washing machine.
- Air dry only.
- Do not squeeze or wring out the brace.
- Do not place in a dryer or near a heater.