

It has come to our attention that some iBreeze Auto CPAP Machines are experiencing “bumpy” fluctuations in pressure, the IPR functionality is not rounding off the pressure changes smoothly and/or some units are even just shutting down. With assistance from the manufacturer we have determined that this is only the case with some of the iBreeze units and not all of them....and that a Software upgrade may be necessary in order to correct these issues. Below are the step-by-step instructions on how to install the software upgrade.

1. Remove the SD Card from the unit.
2. Download the latest iBreeze software to the SD Card.
3. Press the Encoder Button and the top-right button of the iBreeze unit simultaneously to enter physician mode.
4. Change the setting to Service and enter password 1013
5. Now return the SD Card back into the iBreeze unit.
6. Press “Upgrade” and then choose “Yes” to confirm.
7. Wait until the upgrade has completed successfully and you are ready to use your iBreeze unit.