

Trilogy Evo Caregiver checklist



Caregiver name _____

Trainer name _____

The caregiver will be able to: Completed N/A

Apply to all _____

Understand basic indications for Trilogy Evo

Explain the buttons, indicators, and/or connectors on the front, rear and side panels

Front panel

- On/Off (Standby) button
- AC power indicator
- Alarm Silence button/alarm indicator
- Alarm bar
- Touch screen
- Ambient light sensor

Back panel

- Carrying handle
- FI_2 sensor access panel
- Power cord retention clip
- Air vents
- Air inlet

Optional if your device has an oxygen blender module (OBM):

- Oxygen blender
- High pressure oxygen inlet

Patient panel

- Accessory USB port
- Inspiratory port (to patient)
- Proximal pressure port
- Active exhalation valve line connection for Active PAP and Active Flow circuits
- Dual limb active exhalation valve connection (from patient)
- Flow sensor cable connector

Utility panel

- AC power connector
- Air vents
- Low flow oxygen inlet
- Detachable battery access door
- Micro USB port for device service
- Accessory USB port
- Remote alarm or nurse call connector
- DC power connector

The caregiver will be able to: Completed N/A

Apply to all _____

Connect to A/C power or D/C power and explain power source and status

Install the filters


Air-inlet foam filter _____

Optional: particulate filter _____


Assemble and connect the applicable circuit with any ordered accessories (humidifier, adaptors etc)




Passive single limb

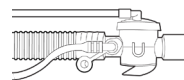

Active PAP

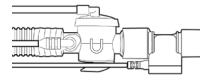

Active Flow

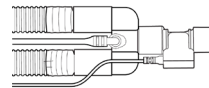

Dual limb

Start Trilogy Evo (Press the on/off standby button)

Explain the user interface

Menu bar

Home

Prescriptions

Options

Status bar



(options and locations will vary with device and settings)

The caregiver will be able to: **Completed** **N/A**

Apply to all

Main window: Workspace

Standby window, prescription window, or monitoring window

Device operation

Select prescription from the main window

Start therapy

Demo different monitoring views

Tap battery indicators to see battery life remaining

Stop therapy (Standby and/or Power Off)

Calibration procedures (in the Options window, if applicable)

Switching prescriptions (if applicable)

Locking and unlocking the screen

The caregiver will be able to: **Completed** **N/A**

Apply to all

Alarms

Setting alarm volume (Options window, Device options, Alarm volume)

Respond to an alarm

Reset an alarm

Alarm and event log (Options window, Alarm & Event Log)

Silence an alarm (temporary)

Cleaning and maintenance

Explain the care, cleaning, and maintenance of Trilogy Evo and applicable accessories

Review emergency procedures

Caregiver signature _____

Date _____

Trainer signature _____

Date _____

