

Goals	Date goals met	Assessor
The clinician will be able to:		
<p>4. Demonstrate how to utilize the provider mode to verify compliance.</p> <p>Specifics:</p> <p><input type="checkbox"/> VIC <input type="checkbox"/> Blower hours</p> <p><input type="checkbox"/> AHI <input type="checkbox"/> System leak</p> <p><input type="checkbox"/> Phone-in <input type="checkbox"/> Periodic breathing</p> <p><input type="checkbox"/> Days>4</p>		
<p>5. Demonstrate steps to adjust Patient settings.</p> <p><input type="checkbox"/> Power the device</p> <p><input type="checkbox"/> Set-up and change device settings</p> <p><input type="checkbox"/> View patient information</p> <p><input type="checkbox"/> Adjustment of flex settings</p> <p><input type="checkbox"/> Adjust humidification</p>		
<p>6. Describe the various methods of obtaining data from the device.</p>		
<p>8. Describe the method and issues involved with adding supplemental oxygen.</p>		
Alert conditions and troubleshooting		
<p>1. Demonstrate understanding of device alarms/alerts, including possible causes, and corrective action.</p> <p>Specific alarms/alerts:</p> <p><input type="checkbox"/> Service required <input type="checkbox"/> SD card</p> <p><input type="checkbox"/> Modem <input type="checkbox"/> Patient reminder</p>		
Cleaning and care		
<p>1. Demonstrate the method of cleaning the device, interface, and replacement of the filters.</p>		

Assessed signature _____ Assessor signature _____

Philips Respirationics is a trademark of Philips Koninklijke Philips N.V. and its affiliates. All rights reserved.

Please visit www.philips.com/respirationics



©2016 Koninklijke Philips N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: US federal law restricts these devices to sale by or on the order of a physician.

e-doc Clay Com EB 6/10/16 MCI 4107614

Philips Respirationics Asia Pacific
+65 6298 1088

Philips Respirationics Australia
+61 (2) 9947 0440
1300 766 488 (toll free, Australia only)

Philips Respirationics United Kingdom
+44 800 1300 845

Philips Respirationics
1010 Murry Ridge Lane
Murrysville, PA 15668 USA

Customer Service
+1 724 387 4000
+1 800 345 6443
(toll free, US and Canada only)

www.philips.com/respirationics