

VISTA TX CERVICAL COLLAR

APPLICATION INSTRUCTIONS:

1. USING A SCOOPING MOTION, SCOOP YOUR CHIN UP INTO THE CHIN CUP.
2. CENTER THE BACK PIECE ON THE BACK OF THE NECK AND ANCHOR INTO PLACE USING THE LATERAL VELCRO FLAPS.

WASHING INSTRUCTIONS

1. This product may be cleaned using a mild detergent and cold water. Do not bleach or use strong detergent. Do not wash in washing machine.
2. Air dry only. Do not squeeze or wring out the brace. Do not place in a dryer or near a heater.